
REF

8090

8095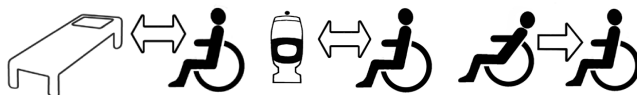

SWL: 200 kg/440 lbs

Table of contents

English	3
Svenska	4
Norsk	5
Dansk	6
Suomi	7
Deutsch	8
Nederlands	9
Français	10
Italiano	11
Español	12
Product information	13

FootStool is used to raise the position of the knees and thereby enable safer, easier bed-to-wheelchair transfer in a seated position. FootStool also provides good support under the feet during visits to the toilet, or when repositioning farther back in a chair. FootStool is made of a material with closed cells, which makes it easy to keep clean.

Check Safety

Visual inspection


Check the condition and function of the assistive device regularly.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.handicare.com.


Placement of FootStool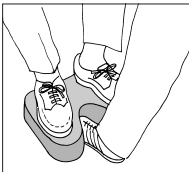

Place the FootStool under the user's feet when the user is sitting on the toilet or in a wheelchair. FootStool is recessed at the front to allow the caregiver to place a foot as close as possible to the stool. The back also has a small recess to facilitate placement close to the toilet.

Using FootStool

FootStool complements other assistive devices such as EasyBelt/FlexiBelt, EasyGlide and SitWalk to facilitate transfers between two locations or for repositioning, for example, farther back in a chair. Since the knees are raised, the thighs are also lifted somewhat, which reduces friction and makes the transfer easier.

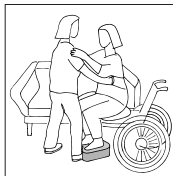Material

Polyethylen

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

Note: Do not subject the product to hard pressure or impact. This may leave indentations in the material.

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com